

NHANES 1999-2000 Second Public Release Dataset

Revised May 2004

Laboratory 06 – Lead, Cadmium, Erythrocyte Protoporphyrin, RBC folate, Serum folate, Vitamin B12, Ferritin, Homocysteine, Methylmalonic Acid, Serum Iron, Total Iron Binding Capacity, Transferrin Saturation, Vitamin A (retinol), Vitamin E (alpha -tocopherol and gamma-tocopherol), Retinyl Palmitate, Retinyl Stearate, Selenium, Total Blood Mercury and Inorganic Mercury, Cotinine, and Urinary Mercury

Description

Erythrocyte Protoporphyrin, RBC folate, Serum folate, Vitamin B12, Ferritin, Homocysteine, Methylmalonic Acid, Serum Iron, Total Iron Binding Capacity, Transferrin Saturation, Vitamin A (retinol), Vitamin E (alpha -tocopherol and gamma-tocopherol), Retinyl Palmitate, Retinyl Stearate, and Selenium

The objectives of this component are: 1) to provide data for monitoring secular trends in measures of nutritional status in the U.S. population; 2) to evaluate the effect of people's habits and behaviors such as physical activity and the use of alcohol, tobacco, and dietary supplements on people's nutritional status; and 3) to evaluate the effect of changes in nutrition and public health policies including welfare reform legislation, food fortification policy, and child nutrition programs on the nutritional status of the U.S. population.

These data will be used to estimate deficiencies and toxicities of specific nutrients in the population and subgroups, to provide population reference data, and to estimate the contribution of diet, supplements, and other factors to serum levels of nutrients. Data will be used for research to further define nutrient requirements as well as optimal levels for disease prevention and health promotion.

Lead

Lead is a known environmental toxin that has been shown to deleteriously affect the nervous, hematopoietic, endocrine, renal and reproductive systems. In young children, lead exposure is a particular hazard because children more readily absorb lead than do adults, and children's developing nervous systems also make them more susceptible to the effects of lead. The primary sources of exposure for children are lead-laden paint chips and dust as a result of deteriorating lead-based paint. The risk for lead exposure is disproportionately higher for children who are poor, non-Hispanic black, living in large metropolitan areas, or living in older housing. Among adults, the most common high exposure sources are occupational.

Blood lead levels measured in previous NHANES programs have been the

cornerstone of lead exposure surveillance in the U.S. The data have been used to document the burden of and dramatic decline of elevated blood lead levels; to promote the reduction of lead use; and to help to redefine national lead poisoning prevention guidelines, standards, and abatement activities.

Cadmium

Cadmium is performed to identify cases of cadmium toxicity. Occupational exposure is the most common cause of elevated cadmium levels.

Total Blood Mercury and Inorganic Mercury

Uncertainties exist regarding levels of exposure to methyl mercury from fish consumption and potential health effects resulting from this exposure. Past estimates of exposure to methyl mercury has been obtained from results of food consumption surveys and measures of methyl mercury in fish. Measures of a biomarker of exposure are needed for improved exposure assessments. Both blood and hair mercury levels will be assessed in two subpopulations particularly vulnerable to the health effects from mercury exposure: children 1-5 years old and women of childbearing age. Women of childbearing age will also have a urine mercury test. Blood measures of total and inorganic mercury will be important for evaluation of exposure from exposure to mercury in interior latex paints.

Cotinine

The specific aims of the component are: 1) to measure the prevalence and extent of tobacco use; 2) to estimate the extent of exposure to environmental tobacco smoke (ETS), and determine trends in exposure to ETS; and 3) to describe the relationship between tobacco use (as well as exposure to ETS) and chronic health conditions, including respiratory and cardiovascular diseases.

The tobacco component for NHANES will include questionnaire items on current and past use of cigarettes, pipes, cigars and smokeless tobacco. Exposure to ETS at home and at work and in-utero ETS exposure among children will also be obtained. ETS exposure will also be assessed for examinees 3 years of age and older through the measurement of serum cotinine, a metabolite of nicotine. In addition, use of nicotine replacement products (e.g., gum and patch) will be collected using questionnaires.

Urinary Mercury

Uncertainties exist regarding levels of exposure to methyl mercury from fish consumption and potential health effects resulting from this exposure. Past estimates of exposure to methyl mercury has been obtained from results of food

consumption surveys and measures of methyl mercury in fish. Measures of a biomarker of exposure are needed for improved exposure assessments. Urinary mercury levels will be assessed in one subpopulation particularly vulnerable to the health effects from mercury exposure.

Eligible Samples

Blood Lead and Cadmium, Erythrocyte Protoporphyrin, Iron, Total Iron Binding Capacity, Ferritin, and Transferrin Saturation

Participants aged 1 year and older who do not meet any of the exclusion criteria

RBC folate, Serum folate, Vitamin B12, Homocysteine, Methylmalonic Acid, Vitamin A (retinol), Vitamin E (alpha -tocopherol and gamma-tocopherol), Retinyl Palmitate, Retinyl Stearate, and Cotinine

Participants aged 3 year and older who do not meet any of the exclusion criteria.

Selenium

Participants aged 3 to 11 years who do not meet any of the exclusion criteria.

Total Blood Mercury and Inorganic Mercury

Participants aged 1–5 years and females aged 16-49 years who do not meet any of the exclusion criteria

Urinary Mercury

Female participants aged 16 to 49 years.

Exclusion Criteria

- Hemophiliacs
- Participants who received chemotherapy within last 4 weeks
- The presence of the following on both arms: rashes, gauze dressings, casts, edema, paralysis, tubes, open sores or wounds, withered arms or limbs missing, damaged, sclerosed or occluded veins, allergies to cleansing reagents, burned or scarred tissue, shunt or IV.

Laboratory

Blood and urine specimens are collected on participants aged one year and older at the mobile examination centers (MECs). Hematological profiles are complete for all participants, and specified laboratory tests are performed upon each specimen based on the participant's age at time of interview and sex.

The laboratory component of NHANES includes the collection, processing, storage, and shipping of blood, urine and other biological and environmental

specimens. The blood collection procedure consists of administering a questionnaire to screen for conditions that excludes the participants from the blood draw and determines fasting status, a blood draw, and collecting specimens for special studies. The urine collection procedure consists of urine collection, urine processing, and pregnancy testing. The Coulter® MAXM performs a complete blood count on blood specimens to provide a study of blood cells and coagulation for all participants.

As part of the overall quality assurance process for the survey, all collection materials, Vacutainer tubes, and storage containers to be used for analyzing trace elements are initially prescreened by the Environmental Health Laboratory Sciences Laboratory for background contamination levels of lead, cadmium, total and inorganic mercury. (Lead, cadmium, and total and inorganic mercury are fairly ubiquitous contaminants; and blood may be collected in ordinary red-top tubes after the acceptability of the test tubes has been confirmed. Special lead-free tubes are not required, however; ordinary EDTA tubes may similarly be used after prescreening has confirmed no contamination.)

Data Collection Methods

Serum and urine specimens are processed, stored, and shipped to the Division of Environmental Health Laboratory Sciences, National Center for Environmental Health, Centers for Disease Control and Prevention for analysis.

Examination Protocol

Detailed specimen collection and processing instructions are discussed in the NHANES Laboratory/Medical Technologists Procedures Manual (LPM). Vials were stored under appropriate frozen (minus 20 degrees Centigrade) conditions until they were shipped to the Division of Environmental Health Laboratory Sciences, National Center for Environmental Health.

Analytic Methodology

Lead and Cadmium

Cadmium and lead are simultaneously measured in whole blood using adaptations of the methods of Miller, et al¹; Parsons, et al² and Stoeppler et al³. Cadmium and lead quantification is based on the measurement of light absorbed at 228.8 nm and 283.3 nm, respectively, by ground state atoms of cadmium and lead from either an electrodeless discharge lamp (EDL) or hollow cathode lamp (HCL) source. Human blood (patient or study) samples, bovine blood quality control pools, and aqueous standards are diluted with a matrix modifier (nitric acid, Triton X-100, and ammonium phosphate). The cadmium and lead contents are determined on a Perkin-Elmer Model SIMAA 6000 simultaneous multi-element atomic absorption

spectrometer with Zeeman background correction.

Erythrocyte Protoporphyrin

Free erythrocyte protoporphyrin (FEP) is measured by a modification of the method of Sassa et al¹. Protoporphyrin is extracted from EDTA-whole blood into a 2:1 (v/v) mixture of ethyl acetate-acetic acid, then back-extracted into diluted hydrochloric acid. The protoporphyrin in the aqueous phase is measured fluorometrically at excitation and emission wavelengths of 404 and 658 nm, respectively. Calculations are based on a processed protoporphyrin IX (free acid) standard curve. After a correction for the individual hematocrit is made, the final concentration of protoporphyrin in a specimen is expressed as micrograms per deciliter of packed red blood cells (g/dL RBC).

RBC folate, Serum folate, and Vitamin B12

Both serum folate and vitamin B12 are measured by using the Bio-Rad Laboratories "Quantaphase II Folate/vitamin B12" radioassay kit². The assay is performed by combining serum or a whole blood hemolysate sample with

¹²⁵I-folate and ⁵⁷Co-vitamin B12 in a solution containing dithiothreitol (DTT) and cyanide. The mixture is boiled to inactivate endogenous folate-binding proteins and to convert the various forms of vitamin B12 to cyanocobalamin. The reduced folate and its analogs are stabilized by DTT during the heating. The mixture is cooled and then combined with immobilized affinity-purified porcine intrinsic factor and folate-binding proteins. The addition of these substances adjusts and buffers the pH of the reaction mixture to 9.2. The reaction mixture is then incubated for 1 hour at room temperature.

During incubation, the endogenous and labeled folate and B12 compete for the limited number of binding sites on the basis of their relative concentrations. The reaction mixtures are then centrifuged and decanted. Labeled and unlabeled folate and vitamin B12, binding to immobilized binding proteins, are concentrated in the bottom of the tube in the form of a pellet. The unbound folate and B12 in the supernatant are discarded, and the radioactivity associated with the pellet is counted. Standard curves are prepared by using the pre-calibrated folate/B12 standards in a human serum albumin base. The concentration of the folate and vitamin B12 in the participant's serum or folate in a participant's whole blood is calculated from the standard curve.

In the erythrocyte folate procedure, the sample is first diluted 1:11 with a solution of 1 g/dL ascorbic acid in water and either incubated for 90 min prior to assay or frozen immediately for later assay. The 90-minute incubation or the freeze-thaw is

necessary for hemolysis of the red blood cells; either allows the endogenous folate conjugates to hydrolyze the conjugated pterylpolyglutamates prior to assay. The sample is further diluted 1:2 with a protein diluent (human serum albumin), resulting in a matrix similar to that of the standards and serum samples.

Ferritin

Ferritin is measured by using the Bio-Rad Laboratories' "QuantImmune Ferritin IRMA" kit³, which is a single-incubation two-site immunoradiometric assay (IRMA)⁴ based on the general principles of assays as described by Addison et al.⁵ and Miles⁶ and modified by Jeong, et al.⁶ In this IRMA, which measures the most basic isoform of ferritin, the highly purified¹²⁵I-labeled antibody to ferritin is the tracer, and the ferritin antibodies are immobilized on polyacrylamide beads as the solid phase. Serum or ferritin standards (made from human liver) are mixed with the combined tracer/solid-phase antibody reagent, and the mixture is incubated. During¹²⁵ incubation, both the immobilized and the¹²⁵I-labeled antibodies bind to the ferritin antigen in the serum or standards, thus creating a "sandwich."

After incubation, the beads are diluted with saline, centrifuged, and decanted. The level of¹²⁵I-labeled ferritin found in the pellets is measured by using a gamma counter. There is a direct relationship between the radioactive levels of the pellets and the amount of endogenous ferritin in the serum or standards, rather than the inverse relationship measured by most radioimmunoassays (RIAs).

Homocysteine

The method is a fluorescence polarization immunoassay (FPIA) from Abbott⁷ Diagnostics. Total homocysteine (tHcy) in plasma is measured by the Abbott Homocysteine assay, a fully automated FPIA method. Dithiothreitol (DTT) reduces homocysteine bound to albumin and to other small molecules, homocysteine, and mixed disulfides, to free thiol. S-adenosylhomocysteine (SAH) hydrolase catalyzes conversion of homocysteine to SAH in the presence of added adenosine. The specific monoclonal antibody and the fluoresceinated SAH analog⁸ tracer constitute the FPIA detection system. Plasma total homocysteine concentrations are calculated by the Abbott IMx® using a machine-stored calibration curve.

As part of ongoing methods comparisons studies, an international round robin⁹ was conducted in 1998. Results obtained using the FPIA method described earlier, were compared to results obtained using high performance liquid chromatography¹⁰ (HPLC) with fluorometric detection at 385 nm excitation and 515 nm emission.

The international round robin demonstrated that the FPIA method was fully equivalent to other frequently used methods (i.e., HPLC-FD, HPLC-ED, and GC/MS). Thus, the Abbott Homocysteine assay was used as primary method for the determination of plasma total homocysteine in NHANES 1999-2000. The HPLC assay was used as a reference method and was performed on a subset of NHANES 1999–2000 specimens for continuing method comparison studies.

Methylmalonic Acid

Methyl malonic acid (MMA) is extracted from plasma or serum along with an added internal standard using a commercially available strong anion exchange resin¹¹. The extracted acid is then derivatized with cyclohexanol to form a dicyclohexyl ester. The derivatized samples are injected onto a gas chromatograph for separation from other constituents. The effluent from the gas chromatograph is monitored with a mass selective detector using selected ion monitoring. Results are quantitated by internal calibration using peak area ratios of MMA and the internal standard (d3MMA).

Iron, Total Iron Binding Capacity, and Transferrin Saturation

Serum iron and total iron-binding capacity (TIBC) are measured by a modification of the automated AAll-25 colorimetric method, which is based on the procedures of Giovanniello¹² and Ramsay¹³. The method has been modified further to be performed on a Alpkem Flow Solutions 3000 (rapid-flow analysis) system. Iron is quantitated by measuring the intensity of the violet complex formed in the reaction between ferrozine and Fe⁺⁺ in acetate buffer at 562 nm. Thiourea is added to complex Cu⁺⁺, which can also bind with ferrozine and yield falsely elevated iron values. In TIBC tests, serum is mixed with 400 g/dL iron solution to saturate the iron-binding sites of the serum transferrin molecules. Magnesium carbonate is used to remove excess iron. Centrifugation is used to precipitate the magnesium carbonate, and the supernatant is measured for iron content.

The transferrin saturation value was calculated as (iron/TIBC) * 100. The iron variable name is LBXIRN and TIBC variable name is LBXTIB.

Vitamin A (retinol), Vitamin E (alpha -tocopherol and gamma-tocopherol), Retinyl Palmitate, Retinyl Stearate

Serum concentrations of vitamins A (retinol) and E (alpha - and gamma-tocopherol), two retinyl esters, and carotenoids are measured using high performance liquid chromatography and multiwavelength detection. A small volume (100 µL) of serum is mixed with an ethanol solution containing two internal standards- retinyl butyrate and apo-8'-carotenal. The analytes are extracted from

the aqueous phase into hexane and dried under vacuum. The extract is redissolved in ethanol and acetonitrile and filtered to remove any insoluble material. An aliquot of the filtrate is injected onto a C18 reversed phase column and eluted with a gradient consisting of ethanol and acetonitrile. Because absorbance of these substances in solution is linearly proportional to concentration, spectrophotometric methods are used for quantitative analysis. Three wavelengths, approximately corresponding to absorption maxima, namely, 300, 325, and 450 nm, are simultaneously monitored and chromatograms are recorded. Quantitation is accomplished by comparing the peak height or area of the analyte in the unknown with the peak height or area of a known amount of the same analyte in a calibrator solution. Calculations are corrected based on the peak height or area of the internal standard in the unknown compared with the peak height or area of the internal standard in the calibrator solution. Retinol and the retinyl esters are compared with retinyl butyrate at 325 nm, alpha- and gamma-tocopherol are compared with retinyl butyrate at 300 nm, and the carotenoids are compared with apo-8'-carotenal at 450 nm.

Selenium

Selenium is measured in serum by atomic absorption spectrometry in a procedure based on the methods described by Lewis et al.¹⁴ by Paschal and Kimberly¹⁵ Quantification is based on the measurement of light absorbed at 196.0 nm by ground state atoms of selenium from a selenium electrodeless discharge lamp (EDL) source. Serum samples, human serum quality control pools, and serum calibration standards are diluted with a matrix modifier (Triton X-100, nickel nitrate, and magnesium nitrate). The selenium content is determined by using a Perkin-Elmer model SIMAA 6000 transversely heated graphite furnace atomic absorption spectrophotometer with Zeeman background correction, or a Model 5100 GFAAS. The Zeeman system offers improved background correction over deuterium arc-corrected systems; use of the latter often results in over-correction^{16, 17} caused by spectral interference from iron or phosphate in the serum.

Low selenium levels have been linked to Keshan's disease. This is a cardiomyopathy that has been observed in young children who live in areas where the soil is low in selenium, particularly in some regions of China. Selenium has been shown to affect drug metabolism and toxicity. In addition, selenium may have a role in cancer prevention.

Total Blood Mercury and Inorganic Mercury

Total mercury in whole blood is measured by flow injection cold vapor atomic absorption analysis with on-line microwave digestion, based on the method by T. Guo and J. Bassner²⁴. Decomposition of organic mercury compounds in blood occurs mainly while the sample (mixed with bromate-bromide reagent and

hydrochloric acid) flows through the digestion coil in the microwave. Further decomposition of organic mercury is achieved by on-line addition of potassium permanganate. The total (organic + inorganic) mercuric mercury released is reduced to mercury vapor by sodium tetrahydroborate. The mercury vapor is measured by the spectrometer at 253.7 nm. Inorganic mercury in whole blood is measured by using stannous chloride as reductant without employing microwave digestion system. Mercury vapor (reduced from inorganic mercury compounds) is measured via the same quartz cell at 253.7 nm. The difference in the total reduced mercury (by sodium tetrahydroborate) and inorganic reduced mercury (by stannous chloride) is taken to represent organic mercury in whole blood . Mercury analysis is performed to identify cases of mercury toxicity. Urinary mercury (total), will also be analyzed on a subset of NHANES subjects using the Perkin-Elmer FIMS.

Cotinine

Cotinine is a major metabolite of nicotine that may be used as a marker for both active smoking, and as an index to Environmental Tobacco Smoke (ETS) exposure, or "passive smoking". Cotinine is generally preferred over nicotine for such assessments because of its substantially longer half-life. The half-life of cotinine in plasma has been estimated to be about 15-20 hrs contrast¹⁸⁻²⁰ ; by contrast, the half-life of nicotine is only 0.5-3 hrs.²⁰⁻²² Cotinine may be measured in serum, urine or saliva -- the half-life of cotinine in all three fluids is essentially the same.¹⁸ Cotinine concentrations tend to be higher (3-8x) in urine than in serum; however, for studies requiring a quantitative assessment of exposure, plasma or serum is regarded as the fluid of choice²³ . Therefore, serum was chosen for NHANES cotinine analyses.

Serum cotinine is measured by an isotope dilution - high performance liquid chromatography / atmospheric pressure chemical ionization tandem mass spectrometric (ID HPLC-APCI MS/MS) method. Briefly, the serum sample is spiked with methyl-D₃ cotinine as an internal standard, and following an equilibration period, the sample is applied to a basified solid phase extraction column. Cotinine is extracted off the column with methylene chloride, the organic extract is concentrated, and the residue is injected onto a short, C18 HPLC column. The eluant from these injections is monitored by APCI-MS/MS, and the m/z 80 daughter ion from the m/z 177 quasi-molecular ion is quantitated, along with additional ions for the internal standard, external standard, and for confirmation. Cotinine concentrations are derived from the ratio of native to labeled cotinine in the sample by comparisons to a standard curve.

Urinary Mercury

Mercury in urine is measured by flow injection cold vapor atomic absorption analysis, which is based on the method that Guo and Bassner developed²⁴. Since mercury in urine is found almost entirely in the inorganic form, Guo and Bassner's method does not use microwave digestion, and decomposition of mercury compounds is achieved by manually adding mixed bromate-bromide reagent and concentrated hydrochloric acid (HCl). Further decomposition of mercury compounds is achieved by adding of potassium permanganate online. The mercury vapor (reduced from inorganic mercury compounds by sodium tetrahydroborate) is measured by the spectrophotometer at 253.7 nm.

Mercury analysis is performed to identify cases of mercury exposure or toxicity. The brain and kidneys are the main organs affected by mercury. Psychic and emotional disturbances are the initial signs of chronic intoxication by elemental mercury vapor or salts. Paresthesias and neuralgia may develop. Renal disease, digestive disturbances, and ocular lesions can also develop. Kidney toxicity is an important consequence of exposure to mercury salts.²⁵

Special Notes for this Dataset

LBXFER – Ferritin

The age cut-off for this assay changed in mid-1999. Prior to 9/8/99 examinees 3 years of age and older were included. Beginning 9/8/99, the age cut-off became 1 year of age and older. Since it is difficult to construct a data file with 2 different age cut-offs, all participants aged 1 year and older are included in this data file. The number of data values for participants under 3 years of age is small. Data users should examine the frequency of values and consider possible exclusion of this age group before analyzing data.

URXUHG – Urinary Mercury

Measures of urinary mercury are assessed on female participants aged 16 to 49 years.

Blood mercury data can be located in the first 1999-2000 release files (June, 2002) on participants 1 to 5 years and female participants aged 16 to 49 years. Hair mercury is located in the NHANES 1999-2000 Public Data File (February, 2003) on participants 1 to 5 years and female participants aged 16 to 49 years.

The analysis of NHANES 1999-2000 laboratory data must be conducted with the key survey design and basic demographic variables. The NHANES 1999-2000 Household Questionnaire Data Files contain demographic data, health indicators, and other related information collected during household interviews. The Household Questionnaire Data Files also contain all survey design variables and sample weights required to analyze these data. The Phlebotomy Examination file includes auxiliary information on duration of fasting, the time of day of the venipuncture, and the conditions precluding venipuncture. The Household

Questionnaire and Phlebotomy Exam files may be linked to the laboratory data file using the unique survey participant identifier SEQN.

References

- 1) Sassa S, Granick JL, Granick S, Kappas A, Levere RD. Microanalyses of erythrocyte protoporphyrin levels by spectrophotometry in the detection of chronic lead intoxication in the subclinical range. *Biochem Med* 1973;8: 135-48.
- 2) Instruction Manual, Bio-Rad Quantaphase II Folate Radioassay Kit. Bio-Rad Laboratories, Hercules, CA: Bio-Rad Laboratories, 1993.
- 3) Instruction Manual, Bio-Rad QuantImmune Ferritin IRMA. Hercules (CA): Bio-Rad Laboratories, 1986.
- 4) Addison G, Beamish M, Hales C, et al. An immunoradiometric assay for ferritin in the serum of normal patients and patients with iron deficiency and iron overload. *J Clin Path* 1972; 25:326.
- 5) Miles L. Measurement of serum ferritin by a 2-site immunoradiometric assay. In: *Handbook of Radioimmunoassay* (Abraham, G ed.). New York: Marcel Dekker, Inc., 1977: Chapter 4.
- 6) Jeong H, Blackmore J, Lewin N. Ferritin immunoradiometric assay, U.S. Patent No. 4,244,940
- 7) Abbott Homocysteine (HCY) assay package insert.
- 8) Shipchandler, MT, Moore EG. Rapid, fully automated measurement of plasma homocysteine with the Abbott IMx® analyzer. *Clin Chem* 1995;41(7):991-4.
- 9) Pfeiffer CM, Huff DL, Smith SJ, Miller DT, Gunter EW. An international comparison of plasma total homocysteine laboratory measurements. *Clin Chem* 1999;45: in press.
- 10) Pfeiffer CM, Huff DL, Gunter EW. Rapid and accurate HPLC assay for total homocysteine and cysteine in a clinical laboratory setting. *Clin Chem*. 1999;45:290-2
- 11) Rasmussen K. Solid-phase sample extraction for rapid determination of methylmalonic acid in serum and urine by a stable-isotope-dilution method. *Clin Chem* 1989;35(2):260-4.
- 12) Giovanniello TJ, Bendetto G, Palmer DW, Peters T. Fully and semiautomated methods for the determination of serum iron and total iron-binding capacity. *J Lab Clin Med* 1968; 71:874.

- 13) Ramsay WNM. The determination of the total iron-binding capacity of serum. *ClinChem Acta* 1957; 2:221.
- 14) Lewis SA, Hardison NW, Veillon C. Comparison of isotope dilution mass spectrometry and graphite furnace atomic absorption spectrometry with Zeeman background correction for determination of plasma selenium. *Anal Chem* 1986;58:1272-3.
- 15) Paschal DC, Kimberly MM. Automated direct determination of selenium in serum by electrothermal atomic absorption spectroscopy. *At Spectrosc* 1986;7:75-8.
- 16) Manning DC. Spectral interferences in graphite furnace atomic absorption spectroscopy. I. The determination of selenium in an iron matrix. *At Absorpt Newslett* 1978;17:107-8.
- 17) Fernandez FJ, Giddings R. Elimination of spectral interferences using Zeeman effect background correction. *At Spectrosc* 1982;3:61-5.
- 18) Jarvis MJ, Russell MAH, Benowitz NL, Feyerabend C. Elimination of cotinine from body fluids: Implications for noninvasive measurement of tobacco smoke exposure. *Am J Public Health* 1988;78:696-8.
- 19) Benowitz NL, Kuyt F, Jacob P, Jones RT, Osman A-L. Cotinine disposition and effects. *Clin Pharmacol Ther* 1983;34:604-11.
- 20) Kyerematen GA, Morgan ML, Chattopadhyay B, deBethizy JD, Vesell ES. Disposition of nicotine and eight metabolites in smokers and nonsmokers: Identification of two metabolites that are longer lived than cotinine. *Clin Pharmacol Ther* 1990;48:641-51.
- 21) Jacob P, Yu L, Wilson M, Benowitz NL. Selected ion monitoring method for determination of nicotine, cotinine and deuterium-labeled analogs: Absence of an isotope effect in the clearance of (S)-nicotine-3',3'-d₂ in humans. *Biol Mass Spec* 1991;20:247-52.
- 22) Armitage AK, Dollery CT, George CF, Houseman TH, Lewis PJ, Turner DM. Absorption and metabolism of nicotine from cigarettes. *Br Med J* 1975;4:313-16.
- 23) Watts RR, Langone JJ, Knight GJ, Lewtas J. Cotinine analytical workshop

report: Consideration of analytical methods for determining cotinine in human body fluids as a measure of passive exposure to tobacco smoke. *Env Health Perspec* 1990;84:173

24)Guo T, Baasner J. Determination of mercury by flow-injection cold vapor atomic absorption spectrometry. *Analytica Chimica Acta* 1993; 278: 189-96.

25)Carson BL, Ellis HV III, McCann JL. Toxicology and biological monitoring of metals in humans. Chelsea (MI): Lewis Publishers, Inc.; 1986: p.130.